



# Harvard TransMatrix


## Transparent matrix and bite registration material based on A-silicones

Harvard TransMatrix allows a transparent reprint of the tooth surface and adjacent teeth. This reprint can be used as a customized transparent matrix for a subsequent restoration (e.g. Veneers) with light-curing composites.

- High final hardness and details reproduction leave the original tooth shape unchanged
- Dimensionally stable during storage and also after disinfection
- Perfect consistency and thixotropy
- **Fast setting time** (1:20 min. in the mouth)
- Pleasant for the patient with neutral smell and taste
- **Time saving** by direct application in the mouth

Additionally, Harvard TransMatrix can be used also as transparent bite registration material.

Restore with transparency.

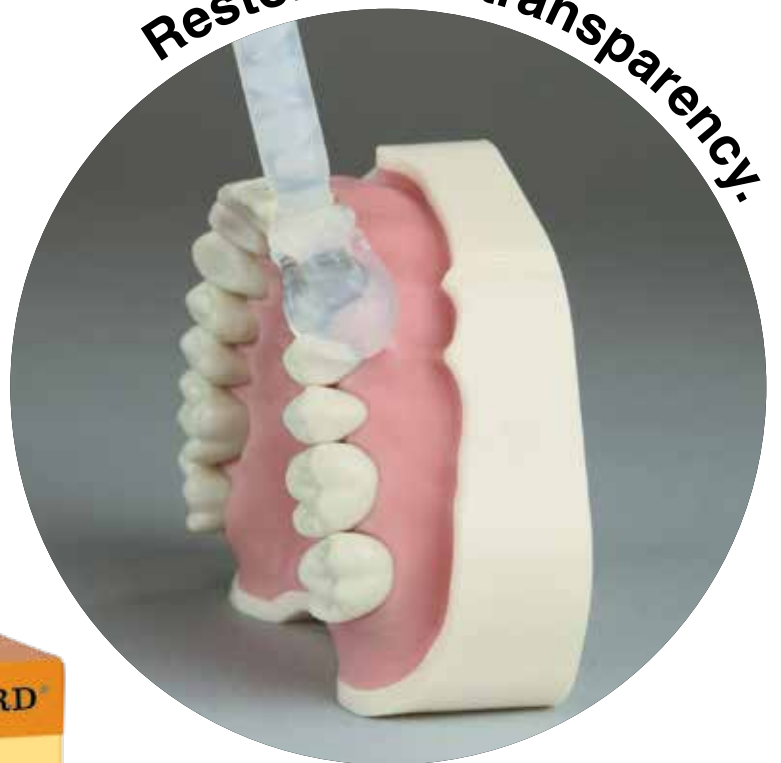

# In 12 steps to success.



1 Initial situation



2 Harvard TransMatrix application on the prepared surface and adjacent teeth



3 Fast setting time (1:20 min. intraoral)



4 Complete matrix



5 Preparation with bevelled enamel



6 Etching of the prepared surface with **Harvard Etch**



7 Application of an adhesive system, e.g. **Harvard InterLock ONE**



8 Application of a slightly heated packable composite in the matrix, e.g. **Harvard UltraFill**



9 Correct repositioning of the filled matrix in the oral cavity



10 40 sec light curing through the matrix. Repeat the process after the removal of the matrix



11 Excess removal and polishing with appropriate rotating instruments and if possible finishing strips



12 Finished direct composite restoration

## Technical Data

Shore A hardness (after 24 hours) 64

## Article information

## Order no.

**Harvard TransMatrix**  
2 x 50 ml Automix cartridge,  
in transparent shade, 12 mixing tips

7083798

**Harvard Auto 1:1 O-Green**  
Refill bag with 50 mixing tips

7096000

**Harvard Dispenser Automix 1:1**

7095100

Liability is excluded for all printing errors and omissions. Before using our Harvard products, the respective directions for use should be noticed in every case. All measurements are internal measurements of Harvard Dental International.

Harvard Distribution Partner.



# HARVARD®

Marke und Qualität seit 1892

Harvard Dental International GmbH  
Margaretenstr. 2 - 4, 15366 Hoppegarten, Germany  
Phone: + 49 (0) 30/99 28 978-0  
Fax: + 49 (0) 30/99 28 978-19  
info@harvard-dental-international.de